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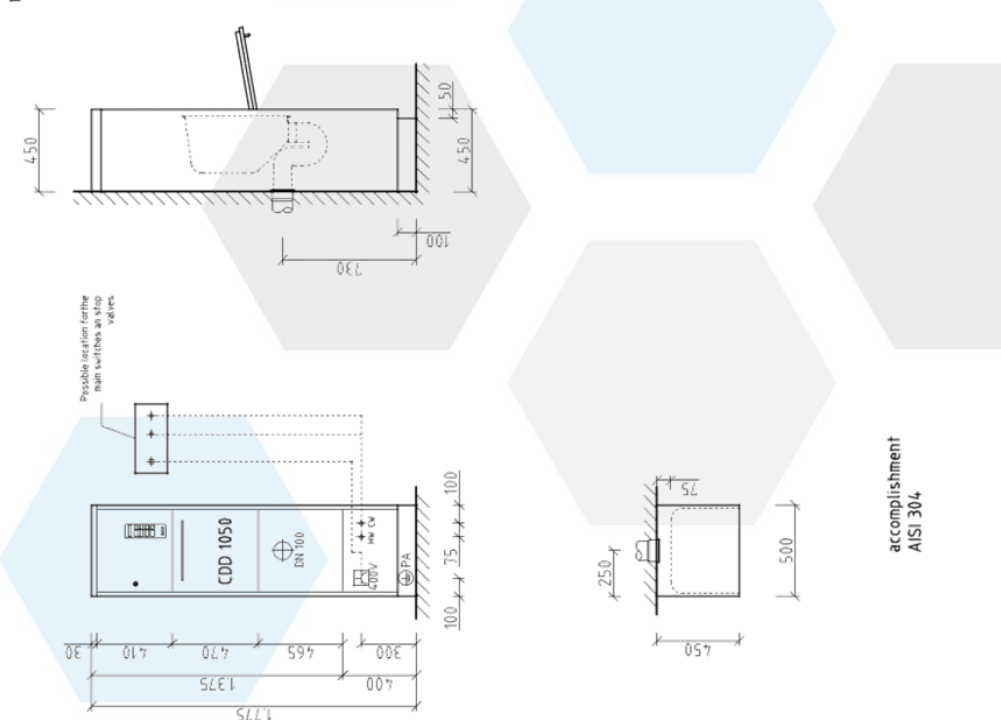
**Geysir CDD 1050
model Basic**

Technical data

HW, CW	provided by the device:	Media connections wall fitting
CW	Cold water connection, recommended 10°DH, 18l/min, 2-5 bar provided by the device: R 3/4" tube, 1.2 m in from the device With union nut, angled filter stop union as ball valve with 3/4" AG flat sealing	
HW	Hot water connection, recommended 10°DH, 18l/min, 2-5 bar, min.40°C provided by the device: R 3/4" tube, 1.2 m in from the device With union nut, angled filter stop union as ball valve with 3/4" AG flat sealing	
Electrical connection	provided by the device: cable on device 5x1.5mm ² , H05-RN, length=0.8m connected load: 4.00V, 3Ph, N/PE, 50Hz, 4.85kW all poles main switch and device junction box, security 10A	
Possible alternative	connected load: 1x 230V, 1 P.-N.-E, 50Hz, 3.35 KW all poles main switch and device part box, hedging 16A	
DN 100	provided by the device: sifon with coupler coupling sleeve DN 100 / NW 110 connecting sleeve DN 100 / NW 110, effluent ca. 60 °C, pH-value 5-9, downpipe rearwards	
PA	potential equalization	

General information:

Water Connections: Armored tubes incl. bag filter are included.
Protective device, for example, electrical insurance, safety valves, water filters 100 µm are on site
Stop valves and main switches are on site at user-friendly body.

This drawing is the property of the company Lischka GmbH and only with their consent totally or partially reproduced, copied or be given to third parties. Subject to change.	release principal	signature	date	index	date	date	description	project-number:
	release LISCHKA GmbH	signature	date	B A	20.12.2011 30.05.2011	Eliminate floor drain Correcting the connections changes	Geysir CDD 1050	construction drawing drawing-number: CDD 1050 - S Date: 03.01.2011
								Mr. von Busch 1.25 to DIN A3